MSD

Animal Health

Instructions for use

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Nobivac.....

ENGLISH

TIME TO CHANGE THE WAY YOU LOOK AT VACCINATION

Congratulations with the chiller unit! This new, innovative appliance is a cooling device exclusively developed to store vaccines for veterinary practices.

The Chiller Unit delivery carton should contain the following components:

- Chiller Unit
- 2 power cords
- 20 cartridges for vaccine vials
- Wall mounting kit
- Template for drilling holes
- Grab tool
- Sheet of magnetic vaccine indicators
- Instruction guide

Please contact your supplier if one of these components is missing.

GENERAL SAFETY INSTRUCTIONS

1. Make sure that the doors shut properly
2. Do not use the Chiller Unit if the unit or the power cable are damaged
3. Never put anything on top of the Chiller Unit, or hang anything on the side of the unit
4. Never remove any fixed internal parts of the Chiller Unit
5. Ensure that the Chiller Unit has been properly wall-mounted before filling it with vaccines
6. Do not modify, remove or add anything to the Chiller Unit which has not been authorized by MSD Animal Health
7. Ensure that the correct vaccine indicators are used for the different vaccines, and whenever you remove a vial, always check the name on its label to ensure you are using the correct vaccine
8. Ensure that you are using and storing the vaccines according to the label directions
9. Young children should be supervised to ensure that they do not play with the Chiller Unit
10. The Chiller Unit is produced and tested to be used with Nobivac vaccines
11. Do not store food, other drugs, or products other than vaccines in the Chiller Unit

PRODUCT SPECIFICATIONS

Product	Chiller Unit, designed to cool and dispense vaccine vials, wall mounted
Gross capacity	Maximum of 195 (3 ml) vials
Voltage	12 volts DC (operating voltage), 100-240 volts AC, 50Hz/60 Hz
Power consumption	Maximum power consumption 85 watts
Quality features	Built-in switching adaptor 100-240 V AC Fits 15 removable cartridges with 13 (3 ml dose) vials each Maintains temperature between 2° and 8° Celsius Magnetic vaccine indicators for Intervet vaccines Front door can be opened for easy loading and cleaning and contains a small door at the bottom for easy dispensing of vials
Dimensions	(LxWxH) 46 x 17 x 30 cm
Unit weight	Maximum unit weight unfilled: 10 kg (22lbs)
Materials used:	Housing: ABS; Cold plate/cartridges/heat sink: Aluminium; Insulation: Polyurethane/Polyfoam
Patent	Patent pending

BASIC FEATURES

LED light:

Green when temperature is between 2° and 8°C. Orange when above 9°C.

Air outlets:

Allow warm air to escape

Window:

May be covered with window shield, included in original packaging

Lock knob:

To lock the front door. Can not be removed; opening or closing by turning.

Bottom door:

To dispense vaccine vials from filled cartridges. Opening by lifting upwards.

Vaccine indicators outside:

Printed magnetic name labels, arranged according to needs of the practice

ENGLISH

HOW TO USE THE CHILLER UNIT

Open the front door

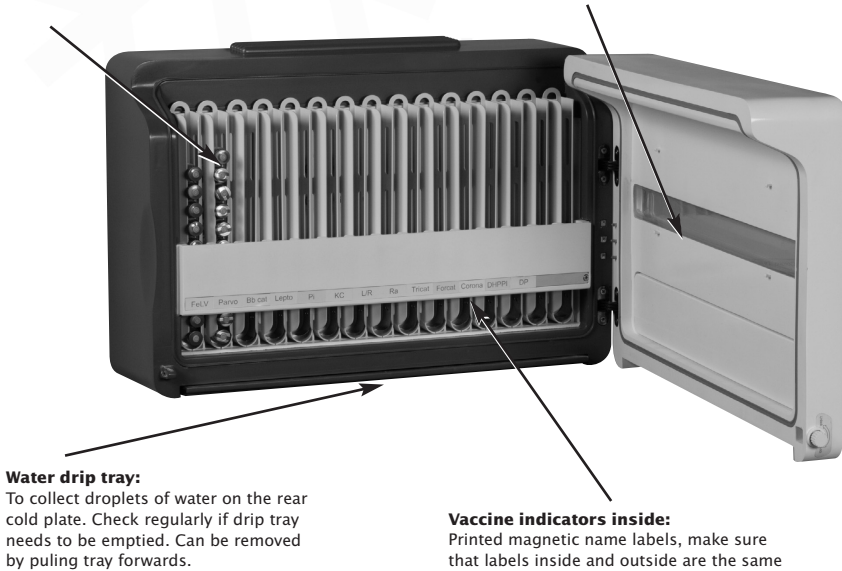
Make sure the lock is unlocked by turning the lock knob.

Cartridges:

To be filled with vaccine vials. Can be removed by moving them upwards.

Window shield:

Can be used to cover window.

Water drip tray:

To collect droplets of water on the rear cold plate. Check regularly if drip tray needs to be emptied. Can be removed by pulling tray forwards.

Vaccine indicators inside:

Printed magnetic name labels, make sure that labels inside and outside are the same

The Chiller Unit can operate in room temperatures between +5° and +35° C.

Check regularly if the drip tray needs to be emptied. Ice crystals may occasionally be found in the tray if humidity is high, but these are no cause for concern.

Cleaning: a periodically cleaning routine will ensure compliance with required hygiene standards. The Chiller Unit can be cleaned just like any other refrigerator. Do not use steel wool, knives or harsh abrasives on any part of the Chiller Unit. Use a sponge with warm water and/or neutral detergent. Rinse the interior and dry with a soft cloth.

INSTALLATION

1. Remove all packaging material. The packaging material is potentially dangerous and must be kept out of reach of children.
2. Plug the power cord into the adaptor at the back of the Chiller Unit. Please be aware that electrical connections must be made in accordance with local regulations.
3. Identify the wall on which you wish to mount the Chiller Unit and ensure that it is capable of supporting the weight of the unit. Some wall types (e.g. plasterboard) may require specialist fixings.
4. Apply the paper template to the wall in the position you wish to mount the Chiller Unit. Use the template to mark the position of the four mounting screws. Make sure the template is level prior to marking the position of the mounting screws. Do not install the unit next to a heating source.
5. Drill holes as appropriate, insert appropriate plugs or wall fixings and screw in four mounting screws. Please check if the accompanying screws are suitable for the walls, use other screws if needed (see point 3).
6. Hang the Chiller Unit securely onto the four screws. Make sure that the air outlets are on the top.

7. Plug the power cable into the power source. If the plug and socket are not the same type, have the socket replaced by a qualified electrician. Do not use extension leads or adaptors.
8. The LED light will initially glow orange; in about 45 minutes the light should turn green once the internal temperature of the Chiller Unit has reached between 2° and 8°C.
9. Open the front door. Make sure that the door is unlocked by turning the lock knob.
10. Remove the cartridges and load them with vaccine vials using the grab tool.
11. Put the loaded cartridges back into the Chiller Unit.
12. Note that after the unit has been opened, the LED light may briefly change to orange until the correct operating temperature is restored.
13. Place the magnetic vaccine indicator strips on the front of the unit and inside the unit, making sure that each vaccine in the unit matches the strips and that the strips inside are matching the strips on the front.

TROUBLESHOOTING GUIDE

1. The chiller unit is not working

- Is there a power failure?
- Is the plug properly inserted in the socket?
- Is the power cord properly connected to the adapter at the back of the chiller unit?

2. The temperature is not between 2° and 8°C

- Are the doors properly closed?
- Is the chiller unit installed next to a heat source?
- Are the air outlets blocked?
- Is the chiller unit running for at least 45 minutes?

3. The LED light is not glowing

- Perform the checks for problem 1

4. Water collects at the vials and at bottom of chiller unit

- Is the water drip tray blocked?
- Are the doors properly closed?

DISPOSAL AND RECYCLING INFORMATION

Dispose according to local, state and federal laws. When this product reaches the end of its life, take it to a collection point designated by local authorities. The separate collection and recycling of products at the time of disposal will help conserve natural resources and ensure that it is recycled in a manner that protects both human health and the environment.

Important information:

Users are cautioned that Intervet will have no obligation with respect to any product that has been improperly installed, adjusted, operated or not maintained in accordance with national and local codes or installation instructions provided with the product, or any product that has its serial number defaced, obliterated or removed, or which has been modified or repaired using unauthorised parts. Intervet is not responsible for any damage as a result from improper use of the Chiller Unit.

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